



THUASNE

MOBIDERM

Hand Bandaging Technique in Reduction Phase



1. Cut the Mobiderm into pieces as shown. Use Mobiderm 5mm x 5mm squares



2. Place the Mobiderm cut outs onto the hand and hold them in place with taping



3. Wrap each finger with adhesive tape - this is optional



4. Bandage over the hand



5. This image above show the completed hand bandage with Mobiderm visible underneath

NOTE:
Depending on severity of oedema - use 5 x 5 mm under compression glove.

0800 486-483
sales@medivex.co.nz
www.medivex.co.nz

NEW ZEALAND DISTRIBUTOR
medivex
HEALTHCARE LIMITED