




THUASNE
COMPRESSIONCARE

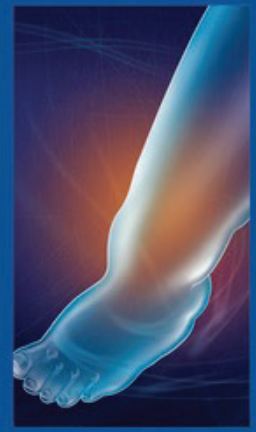

MOBIDERM®

The unique anti-œdema solution

Mobiderm® is a **unique active system** for the **treatment of œdema** of all origins:

lymphœdema, venous œdema, post-traumatic œdema.

At every stage in the treatment of œdema, **Mobiderm®** helps **rapidly reduce** its volume and **soften the tissues**. The system is **quick** and **easy** to apply, **saving time for therapists** and **giving patients greater autonomy**.

THE SYSTEM MOBIDERM®

MOBIDERM® consists of foam blocks of different sizes (5 or 15 mm on each side) inserted between two non-woven layers of material.

It exists in the form of three systems:

- The **bandage**, measuring 10 cm x 3 m
- The **pad**, measuring 27 cm x 1 m
- Made-to-measure **mobilising garments**



THE MECHANISM OF ACTION

MOBIDERM®, combined with **elastic compression** or **non-elastic support**, exerts differentiated pressure on the supporting zone (foam cubes) and its surrounding area.



This pressure, **combined with variations in muscle volume**, creates a shear effect on the thick, solid, indurated subcutaneous tissues.



Cotton bandage



Mobiderm®



Biflexideal®



Biflex®

INDICATIONS :

- lymphœdema / venous œdema
- post-traumatic œdema

CONTRA-INDICATIONS :

- inflammation and skin infections
- very fragile or allergic skin

PRECAUTIONS FOR USE

- place a cotton bandage between Mobiderm® and the skin
- moderate physical exercise to avoid any risk of friction
- monitor, like any bandage

USE CONFIGURATIONS

Reduction phase :

- The **Mobiderm® bandage** is applied over the **Thuasne cotton bandage**, over the whole limb; it is held in place using several layers of **Biflexideal®** bandage during the day.
- Finally, to actively compress the œdema and thereby reduce it effectively, a fourth compression bandage, **Biflex®**, is used for daytime wear only.
- **For more efficacy**, you have to maintain the system by removing one of the short stretch bandages and then **obtain light night time support**.
- Use of **pads for the extremities** (hands/feet) or very **targeted surfaces**, again underneath a reducing bandage.

Maintenance phase :

- Wearing of a **Thuasne Lymphology made-to-measure garment** during the day.
- Self-bandaging at night using **Mobiderm® made-to-measure garments**.



Cotton Bandage



Mobiderm®



Biflexideal®



Biflex®

ADVANTAGES OF MOBIDERM®

- **Improves joint mobility.**
- Significantly **accelerates** the **resorption of œdema** through constant, **efficient drainage**.
- **Softens indurated zones.**
- **Shortens** the duration of **the initial treatment phase**.
- Light, flexible system.
- Can be washed, reused for one patient only.
- Can be prescribed **without any risk of allergic, neurological, orthopaedic or psychological contraindications**.



Before → 5 days → After

MOBIDERM®, applied under a **Thuasne reducing bandage**, leads to a **rapid improvement in skin suppleness** and **reduces œdema**. It saves time for both patients and practitioners, **facilitates out-patient treatment**, **improves patients' compliance with their treatment**, **gives them greater autonomy** and **enhances their quality of life**.